

NIMUE MD RANGE

EDUCATION ONE PAGER

MICROGLIDE GEL | 50ML

A gel-based treatment for use during microneedling treatments to provide a glide medium and introduce HMW and LMW Hyaluronic Acid and Turmeric derived stem cell based active ingredients for hydration and support of skin rejuvenation.

INDICATIONS

- Suitable for use during the microneedling treatment with a roller, microneedling pen device or radio frequency microneedling pen

BENEFITS

- Increases skin hydration
- Improves barrier function and water retention capacity
- Reduces inflammation
- Revitalises and regenerates the skin

KEY INGREDIENTS

- HMW Hyaluronic Acid (*Film-forming, preventing TEWL and increases hydration*)
- LMW Hyaluronic Acid (*Improves internal hydration and skin barrier repair*)
- Turmeric Stem Cells (*Reduces inflammation and tenderness and activates skin remodelling*)

DIRECTIONS FOR USE

MD MICROGLIDE GEL	
Professional use only	
Face, neck and décolleté or treatment area	
3-5ml of Microglide Gel	
Usage guideline	
Area	Pumps
Face, neck and décolleté or treatment area	4ml (18 pumps)
<ul style="list-style-type: none">• Use as a glide medium during microneedling treatments• Apply to a clean, dry skin before and during treatment as needed• Allow to absorb• Follow with aftercare	



MD